

# OneDose Mobile App v1.12.3

## GitHub

### OneDose v1.12.3 Release

1. Patched locked screen after logging out because of the await statements
2. Patched notification querying to allow for sub-agencies to create their own
3. Added colors to folders and protocols
4. Added new EstimationMethods to be customized
  - a. Best Guess (Default)
  - b. FootTevy Formula
  - c. Most Accurate
  - d. APLS Updated
5. Added clean user in async storage because there was always multiple copies of the same one leading to grabbing the incorrect one at the next load
6. Made warnings more visible in medication, electrical, infusion and etc.
7. Added FAST-ED, LAMS, mNIHSS Score, and Oxygen Cylinder Duration Calculator scoring calculators
8. Added accounts by department code and pin instead of subscribing to it

## Apple & Android Notes:

OneDose v1.12.3 – Smarter, Faster, More Intuitive!

### What's New?

Customizable Weight Estimations with multiple methods  
New Scoring Calculators: FAST-ED, LAMS, mNIHSS, Oxygen Cylinder Duration  
Quick Account Setup: Add accounts using a department code & PIN

### Improvements & Fixes

Improved Notifications & Smarter Sign-Out Handling  
Brighter Warnings for medications, electrical, & infusions  
Color-Coded Folders & Protocols for Better Organization

## OneDose Release Notes Page


### Welcome Page 1

```
{  
  id: 1,  
  icon: (
```

```

<FontAwesomeIcon
  icon={faFolderTree}
  size={iconSize}
  color={theme.colors.primary}
/>
),
title: 'Folder & Protocol Coloring',
message:
  'We have added the ability to color folders and protocols.
This will allow you to customize the look of your OneDose.',
},
{
  id: 2,
  icon: (
    <FontAwesomeIcon
      icon={faBuildingShield}
      size={iconSize}
      color={theme.colors.primary}
    />
  ),
  title: 'Subscribe to Different Agencies',
  message:
    'Subscribe to different agencies by department code and
pin and add them to you account.',
},
{
  id: 3,
  icon: (
    <FontAwesomeIcon
      icon={faMobileButton}
      size={iconSize}
      color={theme.colors.primary}
    />
  ),
  title: 'UI Improvements',
  message: 'We have improved the UI of OneDose to be more user friendly.',
},
{
  id: 6,
  icon: (
    <FontAwesomeIcon
      icon={faBroadcastTower}
      size={iconSize}
      color={theme.colors.primary}

```

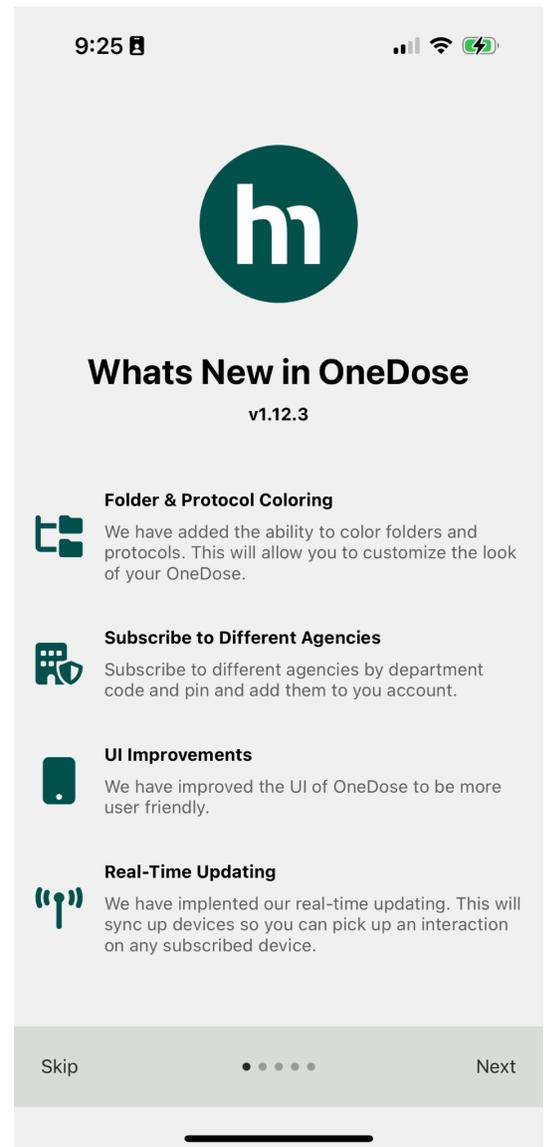


```
    />  
  ),  
  title: 'Real-Time Updating',  
  message:  
    'We have implented our real-time updating. This will sync up devices so you can pick up an  
interaction on any subscribed device.',  
  },
```

## Folder Page 2

### Style Folders & Protocols by Color

We have added the ability to color folders and protocols. This will allow you to customize the look of your OneDose.

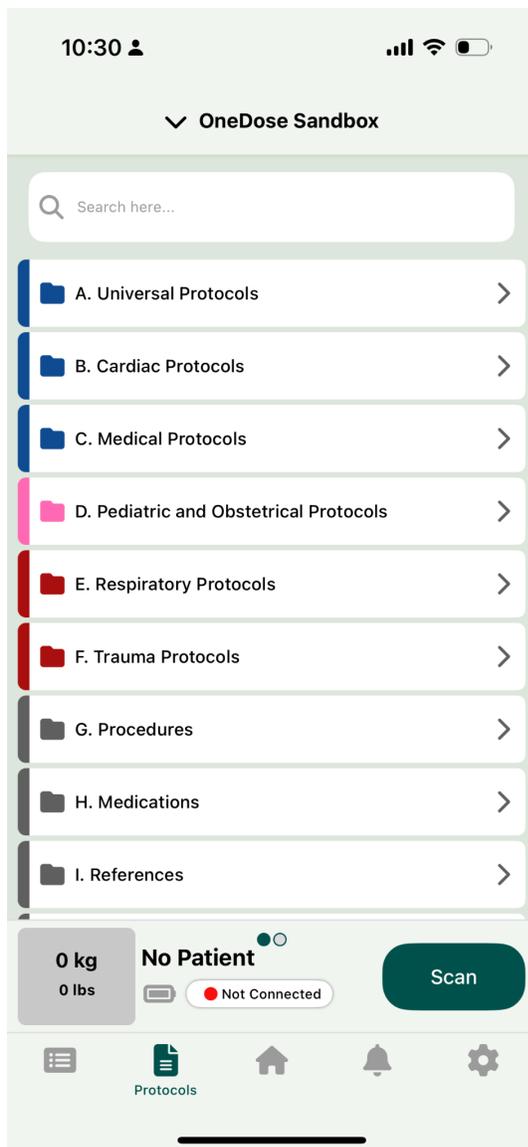



## Add Agencies Page 3

### Subscribe to Different Agencies

You can now subscribe to different agencies by department code and pin and add them to your account. This also allows you to add an agency that does not belong to the same region if you are a OneDose user under more than 1 agency.


### First, Route to the "User Settings"

Select the "Settings" tab in the bottom right corner and then your name at the top of the screen. Then there will be two options, "Subscribe" and "Add".

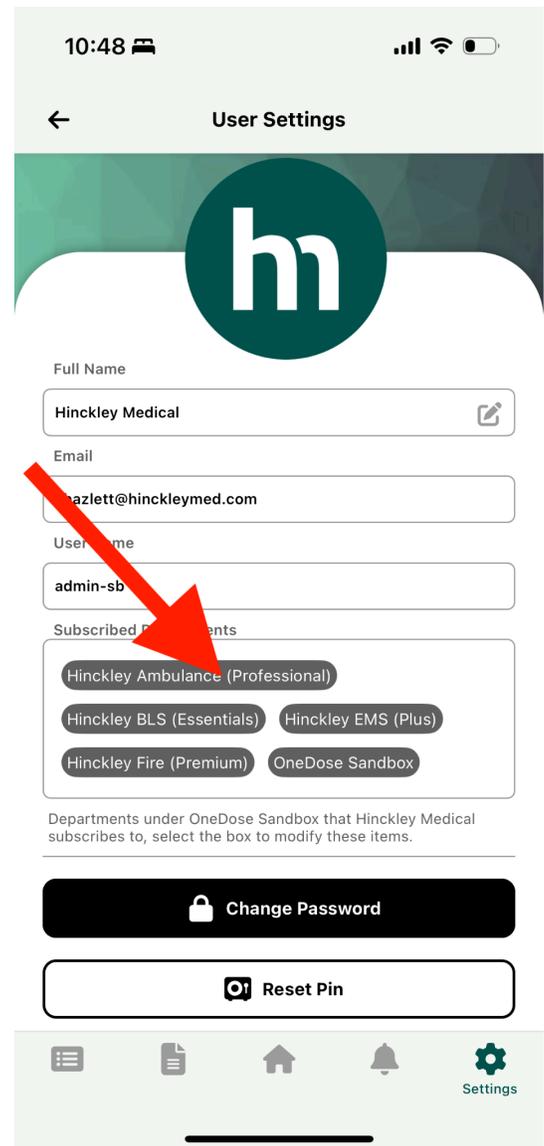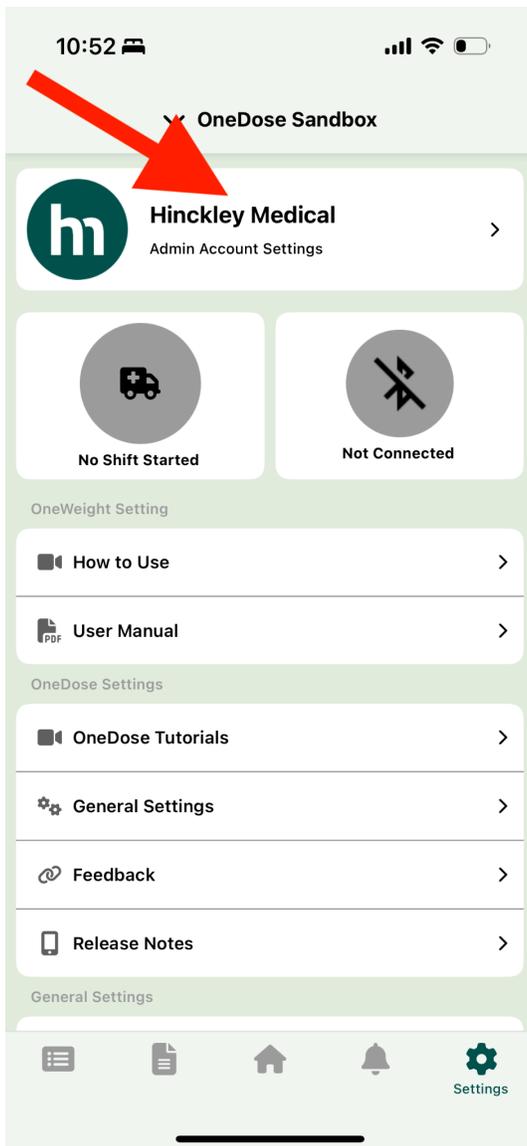


## Subscribe to Agencies

Subscribe to Agencies by selecting them from the list of agencies that populate. Warning depending on your user account permission that decides what agencies you can subscribe to, you may not be able to subscribe to all agencies.

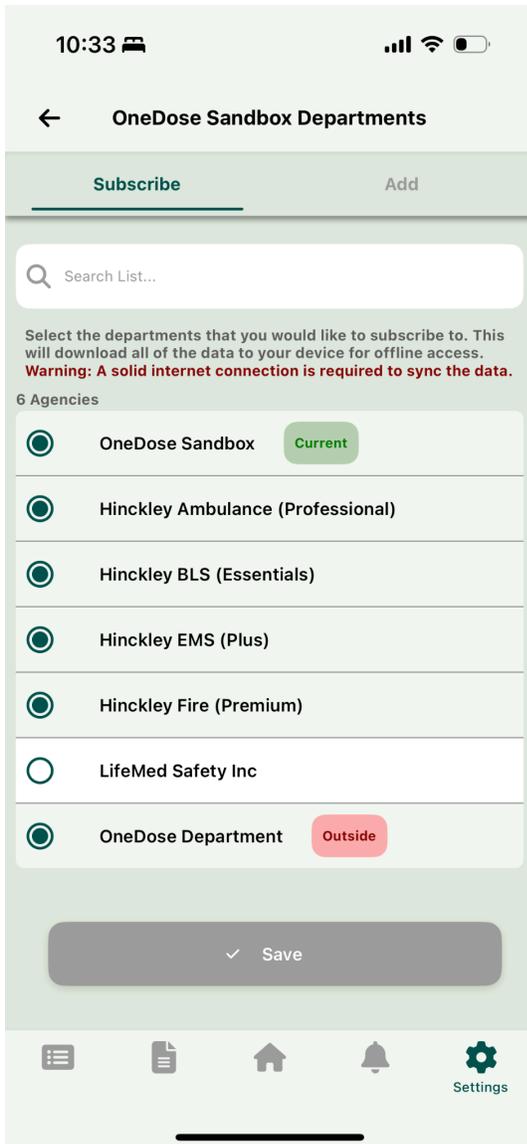

## Add Agencies

Add Agencies by selecting the "Add" tab and then entering the department code and Sign-up pin. Assuming those are valid you will be prompted to add the agency to your account.

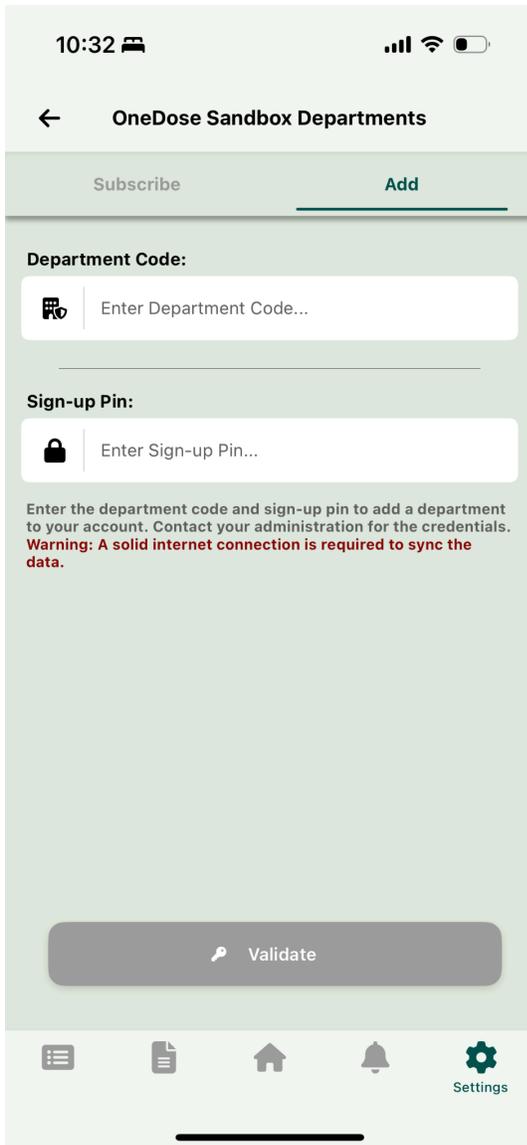

10:32

← OneDose Sandbox Departments

Subscribe **Add**

Department Code:

Enter Department Code...


Sign-up Pin:

Enter Sign-up Pin...

Enter the department code and sign-up pin to add a department to your account. Contact your administration for the credentials.  
**Warning: A solid internet connection is required to sync the data.**

Validate

Settings

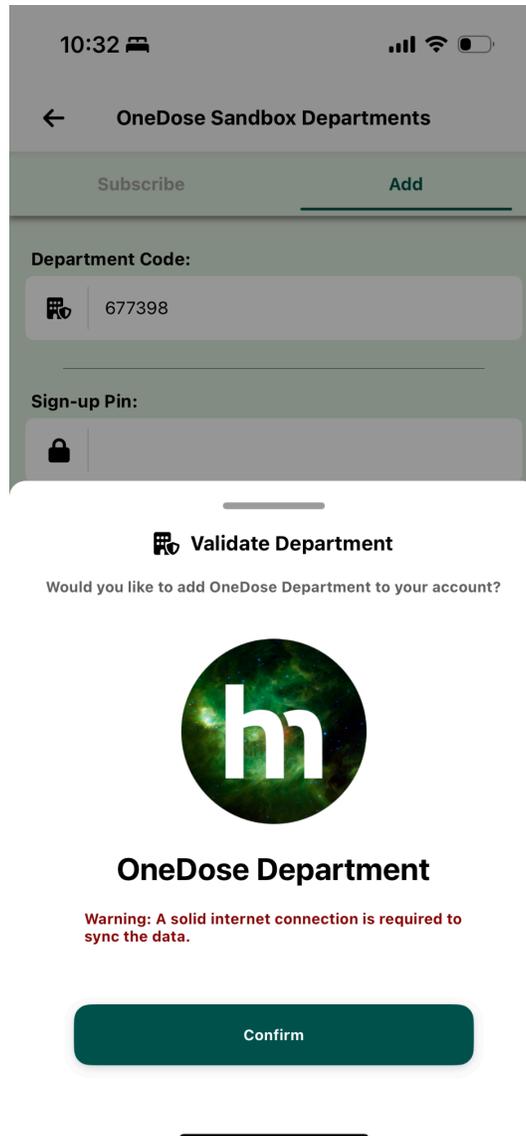

10:32

← OneDose Sandbox Departments

Subscribe **Add**

Department Code: 677398

Sign-up Pin:

**Validate Department**

Would you like to add OneDose Department to your account?



**OneDose Department**

**Warning: A solid internet connection is required to sync the data.**

Confirm

## CPR Assist Page 4


### CPR Assist

You can now use CPR Assist. This is a customizable metronome, timer, and treatment assistant. You can use it to time compressions & ventilation, time track medications, shocks, and more.

**NOTE:** Your agency administration must enable CPR Assist in the admin console to use this feature.

## Start CPR Assist

The first step is to add weight for the patient and then click on the "CPR Assist" tile on the Home Screen.

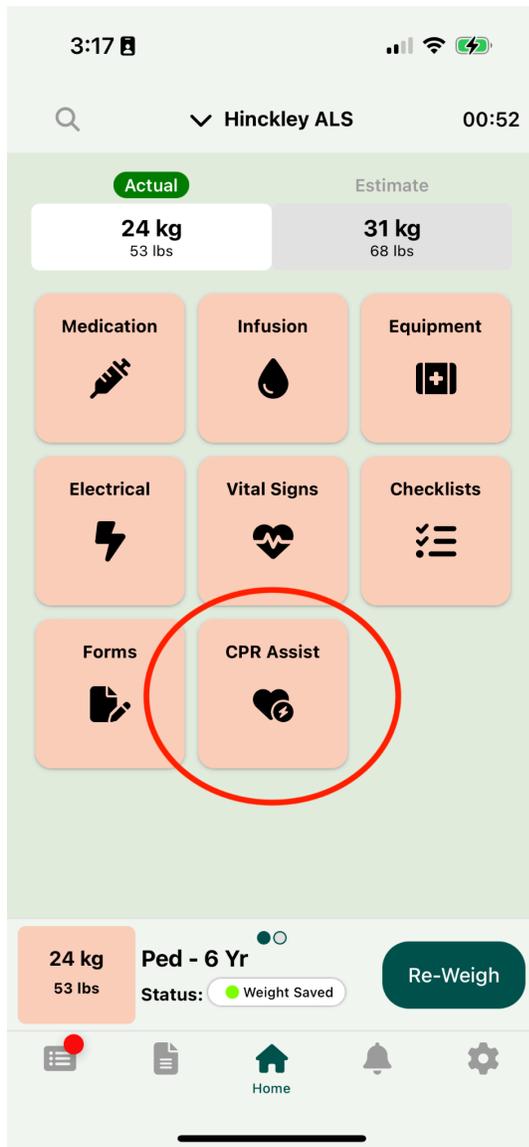

## CPR Assist Layout

CPR Assist will be attached to the top of the screen permanently until it is ended. It contains the main pulse check timer, Epi button & timer, Defibrillation button, and lastly an optional airway device button.


**Persistent Positioning:** CPR Assist now stays attached at the top of the screen, giving you easy access throughout the resuscitation process.

## Key Components:

1. Main Pulse Check Timer: Ensures accurate, timely pulse checks.


2. Epi Button & Timer: Quick access to administer epinephrine with an automatic timer to track dosage intervals
3. Defibrillation Button: Immediate availability for rapid defibrillation actions.
4. Optional Airway Device Button: Adds flexibility with quick access to airway device management if needed.

**Seamless Integration:** All required tools are within reach, letting you focus on patient care with minimal screen interaction!

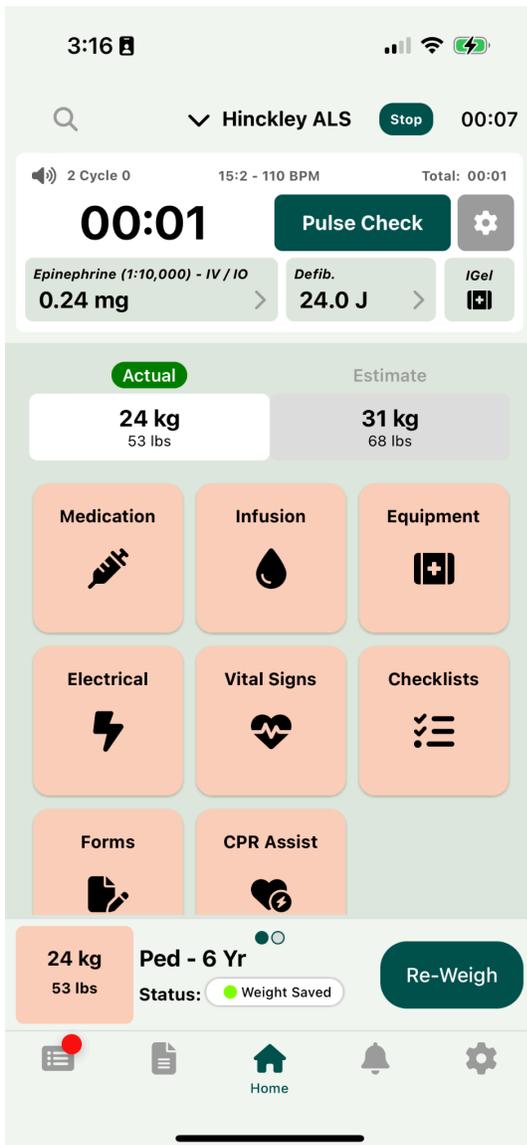

**Fully Customizable:**

Customize CPR Assist to fit your workflow and preferences through our admin console and CPR Assist Settings.

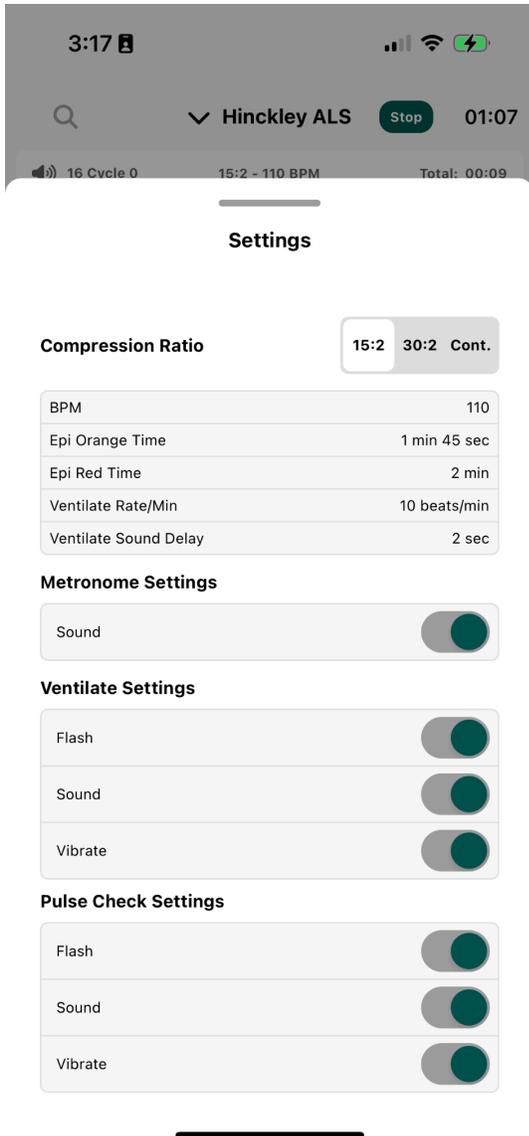

## Realtime Updating Page 5


### Realtime Updating - Beta Mode.

You can now use Realtime Updating in beta mode. This will sync up devices so you can pick up an interaction on any subscribed device.

**NOTE:** Your agency administration must reach out to Hinckley Medical to enable this feature.

## How to use Realtime Updating

The first step is to go to the "Shift" tab in Settings and create a new shift.

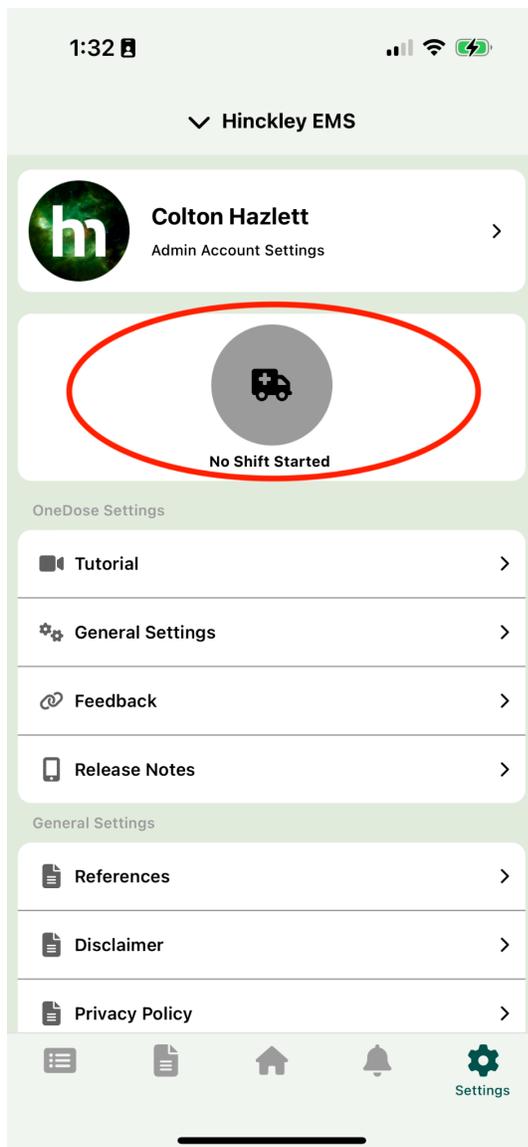

### Realtime Updating Logic

Whenever a user starts an interaction it will reflect across all devices that are subscribed to that shift.

**Requirements:** Realtime Updating requires an internet connection and to be a Premium user.

### Key Components:

1. First step, is to go to the "Shift" tab in Settings and create a new shift.
2. Tag the users that should be synced up in the shift settings.
3. Devices will sync up and you will see a broadcast icon in the top right corner of the screen

- When you start an interaction, it will be broadcasted to all devices subscribed to that shift.

**Admin Viewing:** Admin users can view the shift and the interactions in realtime in the logs screen.

